



Home Page



System Settings

- System Settings
- Design options
- Services
- Auto workflow



DentalManager

- General
- Manufacturing Inbox
- ERP and processing time
- Order import
- Working days
- 3rd party applications
- Milling machines



Site Settings

- This site
- Dentists
- Labs
- Manufacturers
- Operators
- Countries
- Manufacturing process



Tools

- Subscription Management (Google)
- Import/Export
- Download center
- Training Center
- 3Shape Communicate

Select
"Import/Export"



Basic elements

- Materials
- Colors
- Margin line
- Attachments
- 2D Design overlays
- CAD blocks



Anatomy elements

- ScanIt library
- Anatomy and Pontic libraries
- Smile libraries
- Artificial teeth
- Crowns
- Crown Pontics
- Inlays
- Onlays/Veneers
- Temporary Crowns
- Temporary Pontics
- Temporaries on prepared model
- Tabletops



Frame elements

- Copings
- Frame Pontics
- Waxups
- Primary telescopes



Attachments

- Top cap libraries
- Implant systems



Post and Core

- Post and Core systems
- Post and Cores



Bridge elements

- Bars
- Connectors



Full dentures

- Design settings
- Gingiva
- Smile Libraries Chart



Removables

- Frames
- Connectors and Rasts
- Wax profile strips
- Retentions
- Stippled waxes



Digital model

- Digital model design
- Articulator interfaces



Appliances

- Orthodontics Control Panel
- Positioning Guide



Tools > Import/Export

Tasks

Export materials

Import materials

Import materials

To import new materials, click on "Import materials" and browse the desired file (extension ".dmg")
Then select the materials from the list that you want to import.
You can also download the materials directly from the internet.

History Of Import Materials

Export materials

Clone system

Click
"Export materials"

Export materials

- Articulator interfaces
- Artificial teeth
- Attachments
- Bars
- CAD blocks
- Colors
- Connectors
- Copings
- Countries
- Crown Pontics
- Crowns
- Digital model design
- Frame Pontics
- Gingivae
- Implant systems**
- Inlays
- Manufacturing processes
- Margin line
- Materials**
- Onlays/Veneers
- Operators
- Order Form settings
- Order Import
- Positioning Guide
- Post and Core systems
- Post and Cores
- Primary telescopes
- Resin elements

Dental system version: 2019-1

Lock exported materials:

Always overwrite materials with the same IDs when importing:

Delete pre-existing items in destination materials when importing:

Empty Recycle Bin when importing exporting file:

Lock exported materials to Reseller Configuration ID (11):

Type in the version (e.g. 1.0.0.1)

Library creation date: 2020/3/21

Description:

Export to File Export to FTP

Select
"Materials and Manufacturing Processes"
"Materials" or Implant systems.

Export to file